# Dermagiq® Wound

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## Instructions for use

**The Product**
Dermagiq® Wound
15 g ointment for the treatment of superficial wounds

**Ingredients**
Arachis oleum, cera alba, glyceryl oleate, mel, aqua

**Product description**
Dermagiq® Wound is a nourishing ointment for the treatment of superficial wounds. Dermagiq® Wound consists of a base ointment in combination with pure honey. The honey in Dermagiq® Wound contains high levels of honey enzymes, like glucose oxidase. In a moist environment, this enzyme produces low levels of hydrogen peroxide. The high sugar concentration of honey facilitates mixing of wound moisture with Dermagiq® Wound, thus forming an intermediate layer between the wound and the honey, which gives anti-bacterial protection. Dermagiq® Wound does not contain residues from pesticides, herbicides, conservatives or other components that are known to cause allergic reactions.

**Instructions for use**
1. Apply a thin layer of Dermagiq® Wound on the wound

2. The wound can be covered with a wound dressing
3. Treat affected areas 1 to 2 times per day

**Indications**
Dermagiq® Wound can be applied on superficial wounds. Dermagiq® Wound is allowed for all age groups

**Contra indications**
Do not use Dermagiq® Wound if the patient is known to be allergic to honey.

**Side effects**

- During the first minutes after application on minor wounds Dermagiq® Wound can cause a light stinging sensation
- In case of unexpected side effects after use contact your doctor or the manufacturer

**Safety precautions**

- Do not use Dermagiq® Wound if the tube is damaged
- Always replace the top on the tube after use
- Do not allow the tube to come into direct contact with the surface of the wound
- Once the tube has been opened use the contents within 3 months
- Do not treat different patients with the same tube

**Storing conditions**
Store between 4°C and 30 °C

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