Dermagiq® Wound

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Instructions for use

The Product

Dermagiq® Wound
15 g ointment for the treatment of superficial wounds

Ingredients

Arachis oleum, cera alba, glyceryl oleate, mel, aqua, rosa oleum.

Product description

Dermagiq® Wound is a nourishing ointment for the treatment of superficial wounds. Dermagiq® Wound consists of a base ointment in combination with pure honey. The honey in Dermagiq® Wound contains high levels of honey enzymes, like glucose oxidase. In a moist environment, this enzyme produces low levels of hydrogen peroxide. The high sugar concentration of honey facilitates mixing of wound moisture with Dermagiq® Wound, thus forming an intermediate layer between the wound and the honey, which gives anti-bacterial protection. Dermagiq® Wound does not contain residues from pesticides, herbicides or conservatives.

Instructions for use

- Unscrew the white cap from the tube.
- Turn the cap over.
- Open the tube by pressing the tip of the cap through the tube opening.
- Apply a thin layer of Dermagiq® Wound on the wound.
- The wound can be covered with a wound dressing.
- Treat affected areas 1 to 2 times per day.

Indications

Dermagiq® Wound can be applied on superficial wounds. Dermagiq® Wound is allowed for all age groups.

Contra indications

Do not use the product when the patient is known to be sensitive to one of the ingredients of the ointment (see Ingredients). Hypersensitivity to one of the ingredients of the ointment can cause redness, irritation and possibly light pain.

Side effects

- During the first minutes after application, the product can cause a light stinging sensation. The stinging sensation is probably caused by the low pH and high osmolarity of the honey and has no effect on the performance of the product.
- In the event of unexpected side effects after use, contact your doctor or the manufacturer. Side effects can cause inconvenience to the patient.

Safety precautions

- Do not use if the tube is damaged as this can lead to contamination of the product with bacteria and to wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Always replace the top on the tube after use. An open tube could lead to contamination with bacteria and wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not allow the tube to come into direct contact with the surface of the wound. Direct contact between the wound/skin and the uncapped tube could lead to contamination with bacteria and wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not apply this product for more than 30 days continuously. Treatment for more than 30 days continuously could lead to inconvenience to the patient, such as irritation and/or minor pain.
- The contents of the tube must be used within 3 months of opening. Longer use could lead to lower functionality and contamination of the product. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not treat different patients with the same tube as this can lead to cross-contamination with bacteria and to wound infection. Because of the antibacterial protection that honey provides, the chance of wound infection is considered very low.
- Do not use the product after the expiration date as this can lead to lower performance of the product and to inconvenience for the patient
- Do not use sharp objects to open the tube. This can lead to loosening of pieces of the closing membrane and to inconvenience when using the ointment.

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Bfatory Health Products B.V.
NL: Emonla B.V., Drachten – www.emonla.nl
FI: FaunaPharma Oy, Espoo – www.faupharma.fi